


Sling and Hanger Bar Attachment Compatibility Picture Guide

To learn more choosing sling and hanger bar compatibility refer to the Section 5 in the Safe Patient Handling and Mobility: A Toolkit for Program Development 2025 at: <https://www.nvha.net/safe-patient-handling-and-mobility-toolkit/>

Hanger Bar that is designed to be used **only** with slings that have **loop attachment points**

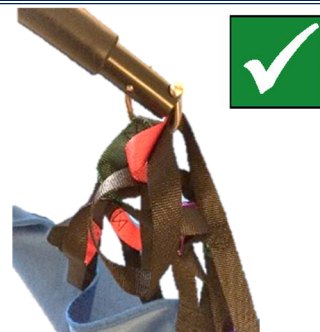

Hanger Bar that is designed to be used **only** with slings that have **key or clip attachment points**



NEVER mix clip/key and loop systems

Slings with keys or clips must **never** be used with loop style hanger bars.

Slings with loops must **never** be used with key or clip style hanger bars.



Make sure sling loops 'fit' securely on a hanger bar connection point and safety latches close properly

